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EN **ATTENTION!** Single Use Product. Do not use if the package is opened or damaged. Gamma radiation sterilized. Do not re-sterilize. Manufacturer recommends single use.

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EN For added security:

- a) Read all instructions contained in this Instruction for Use carefully before using this product.
- b) Ensure that all instructions are stated in the contents of this manual.

This manual was originally written in the Portuguese language.

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1.1 Commercial Models "UN II HI Healing Abutment"

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1.2 Commercial Models "UN II HE Healing Abutment"

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1.3 Commercial Models "UN II Morse Taper Healing Abutment"

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1.4 Commercial Models "UN II HI Emergency Profile Healing Abutment"

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1.5 Commercial Models "UN II HE Emergency Profile Healing Abutment"

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1.6 Commercial Model "UN II Morse Taper Emergency Profile Healing Abutment"

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1.7 Commercial model: "UN II HI ST Healing Abutment"

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1.8 Commercial Model "UN II HE ST Healing Abutment"

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1.9 Commercial Model "UN II Morse Taper ST Healing Abutment"

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1.10 Commercial Model "UN II Morse Taper Cover Healing Abutment"

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1.11 Commercial Model "Stretto Healing Abutment"

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1.12 Commercial Model "Morse Taper Healing Abutment"

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EN 2. Physical principle and fundamentals of product technology, applied to its operation and action

- 3. Options, consumables and support materials
- 4. Special conditions of transport, storage, packaging
- 5. Product Usage Instructions
- 6. Warnings and / or Precautions to be taken
- 7. Indication, Purpose, or Intended Use of the Product
- 8. Undesirable side or side effects and contraindications
- 9. Product Safety and Effectiveness
- 10. Installation or connection to other products
- 11. Product Disposal Concerns
- 12. Symbology
- 13. Warranty term
- 14. Statement of responsibility

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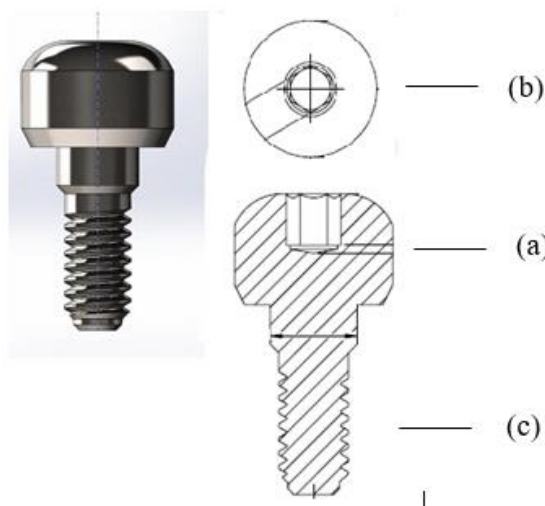
CE
2460

EN 1.1 Commercial Models “UN II HI Healing Abutment”

A) Description: The Implacil UN II HI Healing Abutment consists of a straight abutment, manufactured in commercially pure grade 2 titanium, in accordance with norm ASTM-F67. The Healing Abutments are designed for single use, supplied with collar heights (a) of 1.0mm; 2.0mm; 3.0mm; 4.0mm; 5.0mm; 6.0mm and 7.0mm. Its upper base (b) has a 1.17 hexagon for connection with the insertion driver. Its lower base has an M2.0 thread (c) to be screwed on to the corresponding implant. It offers heights and formats that are compatible with the different abutments available and substitutes them during the cicatrization period, to allow the gengival tissue to obtain the shape required for the desired esthetic profile. The product's diameters relate to the seating platforms for the available implants: 3.5mm; 4.0mm and 5.0mm.

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Dimensions: Ø from base (mm) 3,5; 4,0; 5,0. / Collar Height (mm) 1,0; 2,0; 3,0; 4,0; 5,0; 6,0; 7,0.

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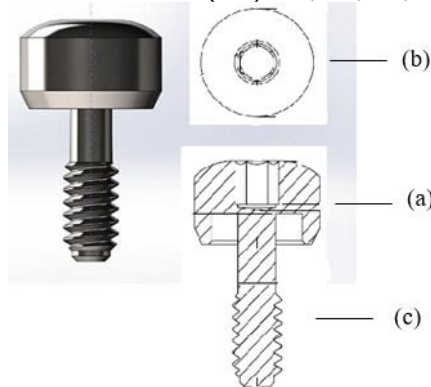
CE
2460

EN 1.2 Commercial Models “UN II HE Healing Abutment”

A) Description: The UN II HE Healing Abutment made by Implacil consists of a straight abutment, manufactured in commercially pure, grade 2 titanium, in accordance with norm ASTM-F67. The Healing Abutments are designed for single use, supplied with collar heights (a) of 1.0mm; 2.0mm; 3.0mm; 4.0mm; 5.0mm; 6.0mm and 7.0mm. Its upper base (b) has a 1.17 hexagon for connection with the insertion driver. Its lower base has an M2.0 thread (c) to be screwed on to the corresponding implant. It offers heights and formats that are compatible with the different abutments available and substitutes them during the cicatrization period. It allows the gengival tissue to obtain the shape required for the desired esthetic profile. The product's diameters relate to the seating platforms for the available implants: 3.5mm; 4.0mm and 5.0mm.

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Dimensions: Ø from base (mm) 3,5; 4,0; 5,0 / Collar Height (mm) 1,0; 2,0; 3,0; 4,0; 5,0; 6,0; 7,0.

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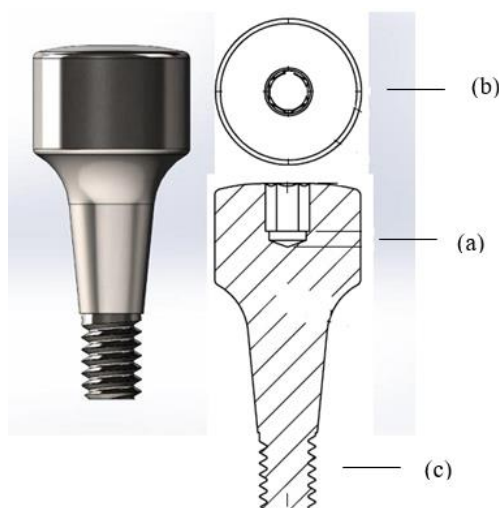
EN 1.3 Commercial Models “UN II Morse Taper Healing Abutment”

A) Description: The UN II Morse Taper Healing Abutment made by Implacil consists of an abutment made in commercially pure, grade 4 titanium, in accordance with norm ASTM F67. The Healing Abutments are designed for single use, supplied with collar heights (a) of 0.80mm; 1.50mm; 3.0mm; 4.5mm and 6.0mm. Its upper base (b) has a 1.17 hexagon for connection with the insertion driver. Its lower base has an M1.8 thread (c) to be screwed on to the corresponding implant. It offers heights and formats that are compatible with the different abutments available and substitutes them during the cicatrization period. It allows the gingival tissue to obtain the shape required for the desired esthetic profile. Its diameters are compatible with all of the diameters for the commercially available Morse Taper implants.

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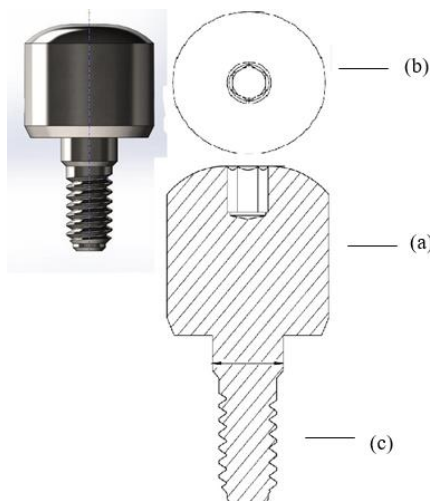
EN 1.4 Commercial Models “UN II HI Emergency Profile Healing Abutment”

A) Description: The UN II HI Emergency Profile Healing Abutment made by Implacil consists of a straight abutment, manufactured in commercially pure, grade 2 titanium, in accordance with norm ASTM-F67. The Healing Abutments are designed for single use, supplied with collar heights (a) of 1.0mm; 2.0mm; 3.0mm; 4.0mm; 5.0mm; 6.0mm and 7.0mm. Its upper base (b) has a 1.17 hexagon for connection with the insertion driver. Its lower base has an M2.0 thread (c) to be screwed on to the corresponding implant. It offers heights and formats that are compatible with the different abutments available and substitutes them during the cicatrization period, to allow the gingival tissue to obtain the shape required for the desired esthetic profile. The product's diameters relate to the seating platforms for the available implants: 3.5mm; 4.0mm and 5.0mm.

B) Accompanying parts and accessories: The product is individually packed. Each package consists of:

- a titanium Healing Abutment

C) Specifications and technical characteristics: Placement Torque 10 Ncm

Dimensions: Ø from base (mm) 5,0; 6,0 / Collar Height (mm) 1,0; 2,0; 3,0; 4,0; 5,0; 6,0; 7,0.

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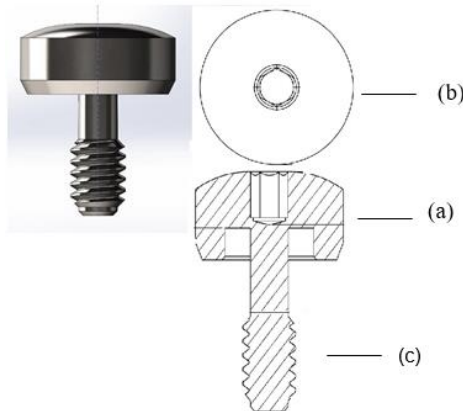
EN 1.5 Commercial Models “UN II HE Emergency Profile Healing Abutment”

A) Description: The UN II HE Emergency Profile Healing Abutment made by Implacil consists of a straight abutment, made in commercially pure, grade 2 titanium, in accordance with norm ASTM-F67. The Healing Abutments are designed for single use, supplied with collar heights (a) of 1.0mm; 2.0mm; 3.0mm; 4.0mm; 5.0mm; 6.0mm and 7.0mm. Its upper base (b) has a 1.17 hexagon for connection with the insertion driver. Its lower base has an M2.0 thread (c) to be screwed on to the corresponding implant. It offers heights and formats that are compatible with the different abutments available and substitutes them during the cicatrization period, to allow the gengival tissue to obtain the shape required for the desired esthetic profile. The product's diameters relate to the seating platforms for the available implants: 3.5mm; 4.0mm and 5.0mm.

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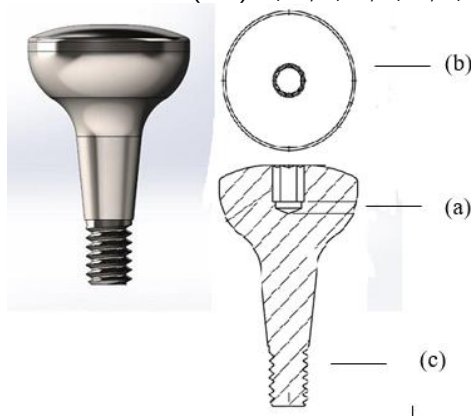
EN 1.6 Commercial Model “UN II Morse Taper Emergency Profile Healing Abutment”

A) Description: The UN II Morse Taper Emergency Profile Healing Abutment made by Implacil De Bortoli consists of a straight abutment manufactured in titanium according to ASTM-F67. The healing abutments are designed for single use, supplied with collar heights (a) of 0.8mm; 1.50mm; 2.50mm; 3.50mm; 4.50mm; 5.50mm and 6.50mm. Its upper base (b) has a 1.17 hexagon for connection with the insertion driver. Its lower base has an M2.0 thread to be screwed to the platform implants 4.0 mm and 5.0 mm, and the lower base thread for 3.5 platform implants is M1.8 (c). It has heights and shapes compatible with the different abutments available, and substitutes them during the healing period to allow the gingival tissue to obtain the shape required for the desired esthetic profile. Its diameters are related to the seating platforms of the commercially available implants: 3.5mm; 4.0mm and 5.0mm.

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Dimensions: Ø of the base (mm): 5.50; 6.30 / Collar height (mm): 0.80; 1.50; 2.50; 3.50; 4.50; 5.50

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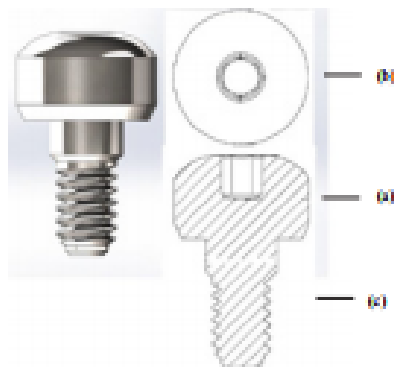
EN 1.7 Commercial model: “UN II HI ST Healing Abutment”

A) Description: The UN II HI ST Healing Abutment made by Implacil De Bortoli consists of a straight abutment manufactured in titanium according to ASTM-F67. The healing abutments are designed for single use, supplied with collar heights (a) of 1.0mm; 2.0mm; 3.0mm; 4.0mm; 5.0mm; 6.0mm and 7.0mm. Its upper base (b) has a 1.17 hexagon for connection with the insertion driver. Its lower base has an M2.0 thread to be screwed to the platform implants 4.0 mm and 5.0 mm, the lower base thread for 3.5 platform implants is M1.8 (c), and when compared to the UN II HI Healing Abutment the thread length is smaller by 1mm. It has heights and shapes compatible with the different abutments available, and substitutes them during the healing period to allow the gingival tissue to obtain the shape required for the desired esthetic profile. Its diameters are related to the seating platforms of the commercially available implants: 3.5mm; 4.0mm and 5.0mm.

B) Accompanying parts and accessories: The product is packaged individually. Each pack consists of:
- one titanium Healing Abutment

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Dimensioni: Ø diametro della base (mm): 3,5; 4,0; 5,0 / Altezza (mm): 1,0; 2,0; 3,0; 4,0; 5,0; 6,0; 7,0

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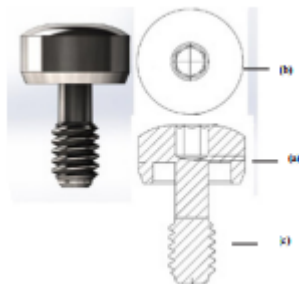
EN 1.8 Commercial Model “UN II HE ST Healing Abutment”

A) Description: The UN II HE ST Healing Abutment made by Implacil De Bortoli consists of a straight abutment manufactured in titanium according to ASTM-F67. The healing abutments are designed for single use, supplied with collar heights (a) of 1.0mm; 2.0mm; 3.0mm; 4.0mm; 5.0mm; 6.0mm and 7.0mm. Its upper base (b) has a 1.17 hexagon for connection with the insertion driver. Its lower base has an M2.0 thread to be screwed to the platform implants 4.0 mm and 5.0 mm, the lower base thread for 3.5 platform implants is M1.8 (c), and when compared to the UN II HE Healing Abutment the thread length is smaller by 1mm. It has heights and shapes compatible with the different pillars available, and it replaces them during the healing period. It allows the gingival tissue to obtain the appropriate shape for the desired esthetic profile. Its diameters are related to the seating platforms of the commercially available implants: 3.5mm; 4.0mm and 5.0mm.

B) Accompanying parts and accessories: The product is packaged individually. Each pack consists of:

- one titanium Healing Abutment

C) Technical specifications and features: Placement Torque: 10 Ncm

Dimensions: Ø of the base (mm): 3,5; 4,0; 5,0 / Collar height (mm): 1,0; 2,0; 3,0; 4,0; 5,0; 6,0; 7,0

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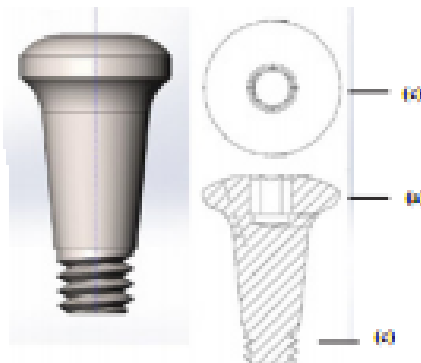
EN 1.9 Commercial Model “UN II Morse Taper ST Healing Abutment”

A) Description: The UN II Morse Taper ST Healing Abutment made by Implacil De Bortoli consists of a straight abutment manufactured in titanium according to ASTM-F67. The healing abutments are designed for single use, supplied with collar heights (a) of 0.80mm; 1.50mm; 2.50mm; 3.50mm; 4.50mm and 5.50mm. Its upper base (b) has a 1.17 hexagon for connection with the insertion driver. Its lower base has an M1.8 (c) thread to be screwed to the corresponding Implant, and when compared to the UN II Morse Taper Healing Abutment the thread length is smaller by 1mm. It has heights and shapes compatible with the different pillars available, and it replaces them during the healing period. It allows the gingival tissue to obtain the appropriate shape for the desired esthetic profile. Its diameters are compatible with all diameters of the commercially available Morse Taper implants.

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Dimensions: Ø of the base (mm): 3,50; 4,50 / Collar height (mm): 0,80; 1,50; 2,50; 3,50; 4,50; 5,50

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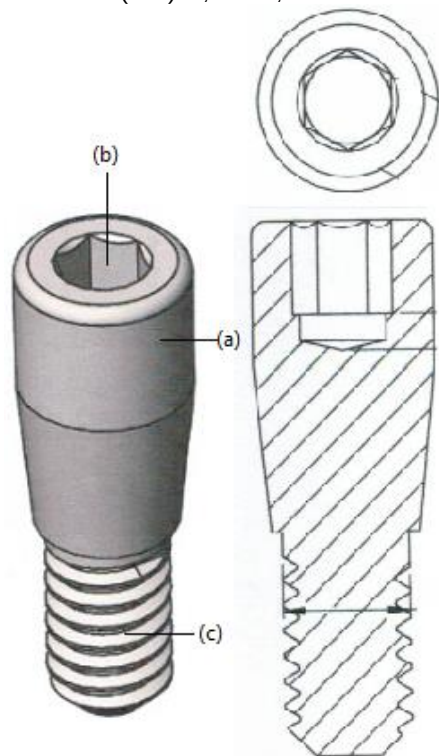
EN 1.10 Commercial Model “UN II Morse Taper Cover Healing Abutment”

A) Description: The UN II Cover Cone Morse Scar consists of a post made of titanium according to ASTM-F67. The healers are single use, presented with strap heights of 1.00mm and 2.00mm. In its upper base (b) has a hexagon of 1.17 for connection with the placement key. At its bottom base it has a M1.8 (c) thread to be threaded to the corresponding Implant. Replaces the abutments during the healing period. It allows the gingival tissue to obtain the shape appropriate to the desired aesthetic profile. Their diameters are compatible with all commercially available Morse Cone implant diameters.

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Dimensions: Ø of the base (mm): 2,50 / Collar height (mm): 1,00; 2,00.

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EN 1.11 Commercial Model “Stretto Healing Abutment”

A) Description: Implacil De Bortoli's Stretto Healer consists of a pillar, made of titanium, in accordance with ASTM-F67. Healers are for single use, presented with strap heights of 0.80mm; 1.50mm; 2.50mm; 3.50mm; 4.50mm; and 5.50mm. In its upper base it has a 1.17 hexagon for connection with the placement key. In its lower base it has an M1.4 thread to be threaded to the corresponding implant. Replaces abutments during the healing period. It allows the gingival tissue to obtain the appropriate shape for the desired aesthetic profile.

Its diameters are compatible with all implant diameters from the Stretto line commercialized.

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- a Stretto Healer available in pink.

C) Specifications and technical characteristics: Placement Torque: 10 Ncm

Dimensions: Base Ø (mm): 3,0; / Strap height (mm): 0,80; 1,50; 2,50; 3,50; 4,50; 5,50.

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EN 1.12 Commercial Model “Morse Taper Healing Abutment”

A) Description: The Implacil De Bortoli Morse Taper Healing Abutment consists of a pillar, manufactured in titanium alloy, in accordance with ASTM-F136. Healers are for single use, presented with strap heights of 0mm, 1mm and 2mm. In its upper base it has a 1.17 hexagon for connection with the placement key. In its lower base it has an M1.8 thread to be threaded to the corresponding implant. Replaces abutments during the healing period. It allows the gingival tissue to obtain the appropriate shape for the desired aesthetic profile. Their diameters are compatible with all commercialized Morse Cone implant diameters.

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Dimensions: Base Ø (mm): 2,5; / Strap height (mm): 0, 1, and 2

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EN 1.13 Commercial Model “Morse Taper ST Healing Abutment”

A) Description: The Implacil De Bortoli Morse Taper ST Healing Abutment consists of a pillar, manufactured in titanium alloy, in accordance with ASTM-F136. Healers are for single use, presented with strap heights of 0.00mm. In its upper base it has a 1.17 hexagon for connection with the placement key. In its lower base it has an M1.8 thread to be threaded to the corresponding Implant, compared to the Healing Cone Morse, they have a thread length 1mm shorter. It has heights and formats compatible with the different abutments available, and replaces them during the healing period. It allows the gingival tissue to obtain the appropriate shape for the desired aesthetic profile. Its diameters are compatible with all diameters of Cone Morse type implants with lengths of 5 and 6 mm sold.

B) Accompanying parts and accessories: The product is individually packaged. Each pack is made up of:

- a titanium healer

C) Specifications and technical characteristics: Placement Torque: 10 Ncm.

Dimensions: Base Ø (mm): 2,5; / Strap height (mm): 0.00.

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EN Physical principal and technological fundaments for the product, applied for its functionality and action: Implacil Healing Abutments are mechanical components used to model the gingival tissue during the period of waiting before the implant is fitted. They are fixed to the implant using a screw. They are also indicated for protecting Implacil Implants, preventing the soft tissue from closing over the implant and promoting a perfect adaptation of the previously selected prosthetic abutments.

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EN Optional extras, consumer materials and support materials: Optional extras: Nonapplicable. Consumer Materials: Non-applicable. Support material: Instructions for use. Manufactured items for exclusive use with the Company's products: All of the components described in this Instruction for Use are for exclusive use with the Healing Abutment product. Attention: The utilization of any component, accessory or material not specified or provided for in these instructions for use is the sole responsibility of the user.

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EN Special conditions for product transport, storage and packing:

Store at room temperature in a cool place away from light and heat. Avoid exposure to heat and excessive humidity. Do not exert excess load on the packaging to avoid compromising the seals that guarantee the sterility of the product.

Implacil De Bortoli Universal II Scars should be shipped in a clean, dry and at room temperature. Transport should be carried out properly to avoid falling and damage to its original packaging and mechanical shocks that could damage the product.

Implacil De Bortoli UNII Healing should only be used by specialists in the field. A conscientious program of home oral hygiene and regular professional attention are required to maintain restorations in good condition.

Implacil De Bortoli UNII Healers are designed for single use and reprocessing is prohibited. Do not use it at all if the packaging is broken.

The professional must observe the force exerted at the time of application of the product so as not to damage it and inform the patient of the need for periodic monitoring that avoids physical and mechanical efforts in order not to subject the product to undue stress.

The end result of application of the product may not be successful in cases where the technique used is not appropriate and loss or fracture of the product may occur. Incorrect application of the product can have side effects in the region where it was applied such as pain, swelling, tenderness, tissue reaction and infection.

The use of Implacil De Bortoli UNII Scars is contraindicated in cases where the patient has chronic periodontal inflammation, oral hygiene and inadequate parafunctional habits, active intra-oral infection and in Immediate Load cases where primary implant stability is not. is appropriate.

ATTENTION !!!

Single Use Product.

Do not use if packaging is damaged or damaged.

Sterile product. Do not resterilize. Sterilized by gamma radiation.

Forbidden to reprocess. Manufacturer Recommends Single Use.

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Envelope Tyvek – código 0332

EN Instructions for use of the product:

- Implacil Healing Abutments come in a blister pack sealed with Tyvek. The blister is placed inside a cardboard carton.
- Check that the packaging is in perfect condition. Do not use the product if the packaging is damaged.
- Open the outer packaging. Remove the blister from the box and check that it is perfectly sealed.
- Remove the Healing Abutment from the blister pack.
- Screw it onto the implant.

The product is presented in individual packaging.

Package content:

01 Universal Healing II

03 Traceability labels with the identification of the product for mandatory fixation: in the clinical chart, in the document to be delivered to the patient, and in the tax documentation that generates the charge, according to the model below:

IMPLACIL DE BORTOLI Produto / Product:		Implacil De Bortoli - Material Odontológico S.A. Reg. ANVISA: xxxxxx Resp. Tec.: Nilton De Bortoli Junior CRC: 32160 Uso restrito a profissionais habilitados
 :		
 :		
 :		
Registro ANVISA nº: 80259860031		

This instruction for use will be available on the Implacil De Bortoli website (www.implacildebortoli.com.br) in accordance with current legislation on the provision of non-printed health product instruction - Normative Instruction No. 4 of 15 June 2012. Printed instruction can be requested through the Customer Service - 0800.779.0011 and sent at no additional cost.

To locate it, check the “IFU” number and version of the instruction that is indicated on the Purchased Product Label.

To read the instructions for use on the site you need the program Adobe Acrobat Reader.

User Alert: Always note the correlation of the version of the instructions for use indicated on the label of the purchased product as made available by Implacil De Bortoli.

Kind of packing:

The material and type of packaging used to package Universal II Implacil De Bortoli Scars are well known and universally used for both medicines and health products and are on the ANVISA Packaging Table.

Packaging Features and Dimensions:

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EN Warnings and/or precautions to be taken: The utilization of Implacil Healing Abutments must only be carried out by specialized dental surgeons. A conscientious routine of oral hygiene at home and regular professional treatment are required in order to conserve the restorations in good condition. Implacil Healing Abutments are designed for single use and their re-utilization is strictly prohibited. Under no circumstances must they be used if the packing seal is broken. The dental surgeon must pay attention to the amount of force used when applying the product so as not to damage it and must also inform the patient of the need for periodic monitoring that avoids physical and mechanical effort in order to avoid subjecting the product to undue strain, as well as the need for proper hygiene.

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EN Indication, purpose or use to which the product will be put: Implacil Healing Abutments are indicated for the cicatrization of the soft tissue (gums) protecting Implacil Implants, preventing the soft tissue from closing over the implant and promoting a perfect adaptation of the previously selected prosthetic abutments, while waiting for the prosthetic component to be made.

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EN Undesired secondary or side effects and contraindications: The end result of the application of the product may be unsuccessful if inappropriate techniques used and this may lead to loss or fracturing of the product. Incorrect application of the product may lead to side effects in the region where it is applied, such as: pain, swelling, sensitivity, tissue reaction and infection. The utilization of Implacil Healing Abutments is contraindicated in cases where the patient presents chronic periodontal inflammation and inadequate oral hygiene and parafunctional habits, active intra-oral infection and in cases of Immediate Loading where the primary stability of the implant is insufficient.

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EN Product safety and efficiency: Implacil Healing Abutments are totally safe, provided that these instructions for use are followed in all of the procedures carried out. This product is manufactured using non-toxic medical-odontological. When stored and conserved in accordance with the recommendations for use, the product characteristics will suffer no alterations or modifications, always respecting the shelf life of the same.

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EN Installation or connection to other products: Implacil Healing Abutments are connected to previously installed and osseointegrated implants, according to the compatible models for each type of connection. The utilization of any component, accessory or material not specified or provided for in these instructions for use is the sole responsibility of the user.

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EN Care to be Taken in Disposal of Product: To avoid environmental contamination or inappropriate use of the Healing Abutments, after utilization they must be disposed of in a suitable location, in other words, in a place for potentially contaminated materials.

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EN Term of Guarantee: IMPLACIL DE BORTOLI - MATERIAL ODONTOLOGICO S.A. guarantees this product against any material or manufacturing defect that may appear within 365 days, counting from the original date of purchase. Provided it is suitably transported, stored and packaged in accordance with these instructions for use, the product will have a shelf life of 2 years.

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