PORTUGUÊS

Instrumentais Cirúrgicos Rotativos

INGLÊS

Rotary Surgical Instruments

ESPAÑHOL

Instrumentos Quirúrgicos Rotativos

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Revisão: 10

PRODUCT

Technical Name: DENTAL SURGICAL INSTRUMENTS

Commercial Name: ROTARY SURGICAL INSTRUMENTS

ATTENTION!!!

DO NOT USE IF THE PACK HAS BEEN TAMPERED WITH OR DAMAGED. NON-STERILE PRODUCT. STERILIZE BEFORE USE. REUSABLE. CLEAN AND STERILIZE BEFORE REUSE.

Commercial Models:

25089 - IMPLANT DRIVER FOR MOTOR HE 3.5
25096 - IMPLANT DRIVER FOR MOTOR HE 4.0
20152 - IMPLANT DRIVER FOR MOTOR HI 3.5
24693 - IMPLANT DRIVER FOR MOTOR HI 4.0
19651 - DRILL EXTENSION DRIVER
218634 - AR MORSE TAPER INSERTION DRIVER FOR MOTOR
28664 - DRIVER MOTOR FOR GRAFT SCREW
28665 - DRIVER INSERTION HE MOTOR 3.3
28666 - DRIVER INSERTION HE MOTOR 3.5
28667 - DRIVER INSERTION HE MOTOR 3.75
28668 - DRIVER INSERTION HE MOTOR 4.0
28669 - DRIVER INSERTION HI MOTOR 3.3
28670 - DRIVER INSERTION HI MOTOR 3.5
28671 - DRIVER INSERTION HI MOTOR 3.75
28672 - DRIVER INSERTION HI MOTOR 4.0
28673 - DRIVER INSERTION HI/CM MOTOR 3.3
28674 - DRIVER INSERTION HI/CM MOTOR 3.5
28675 - DRIVER INSERTION HI/CM MOTOR 3.75
28676 - DRIVER INSERTION HI/CM MOTOR 4.0
28677 - DRIVER INSERTION HE ST MOTOR 3.3
28678 - DRIVER INSERTION HE ST MOTOR 3.5
28679 - DRIVER INSERTION HE ST MOTOR 3.75
28680 - DRIVER INSERTION HE ST MOTOR 4.0
28681 - DRIVER INSERTION HI ST MOTOR 3.3
28682 - DRIVER INSERTION HI ST MOTOR 3.5
28683 - DRIVER INSERTION HI ST MOTOR 3.75
28684 - DRIVER INSERTION HI ST MOTOR 4.0
28685 - DRIVER INSERTION HI/CM ST MOTOR 3.3
28686 - DRIVER INSERTION HI/CM ST MOTOR 3.5
28687 - DRIVER INSERTION HI/CM ST MOTOR 3.75
28688 - DRIVER INSERTION HI/CM ST MOTOR 4.0
28689 - AR DUE CONE MORSE TAPER INSERTION DRIVER FOR MOTOR
28690 - DRIVER INSERTION HE MOTOR LONG 3.3
28692 - DRIVER INSERTION HE MOTOR LONG 3.5
28693 - DRIVER INSERTION HE MOTOR LONG 3.75
28694 - DRIVER INSERTION HE MOTOR LONG 4.0
28695 - DRIVER INSERTION HI MOTOR LONG 3.3
28696 - DRIVER INSERTION HI MOTOR LONG 3.5
28697 - DRIVER INSERTION HI MOTOR LONG 3.75
28698 - DRIVER INSERTION HI MOTOR LONG 4.0
28701 - DRIVER INSERTION HI/CM MOTOR LONG 3.3
28702 - DRIVER INSERTION HI/CM MOTOR LONG 3.5
28703 - DRIVER INSERTION HI/CM MOTOR LONG 3.75
28704 - DRIVER INSERTION HI/CM MOTOR LONG 4.0

28705 - DRIVER INSERTION HE ST MOTOR LONG 3.3
28706 - DRIVER INSERTION HE ST MOTOR LONG 3.5
28707 - DRIVER INSERTION HE ST MOTOR LONG 3.75
28708 - DRIVER INSERTION HE ST MOTOR LONG 4.0
28709 - DRIVER INSERTION HI ST MOTOR LONG 3.3
28710 - DRIVER INSERTION HI ST MOTOR LONG 3.5
28711 - DRIVER INSERTION HI ST MOTOR LONG 3.75
28712 - DRIVER INSERTION HI ST MOTOR LONG 4.0
28713 - DRIVER INSERTION HI/CM ST MOTOR LONG 3.3
28714 - DRIVER INSERTION HI/CM ST MOTOR LONG 3.5
28715 - DRIVER INSERTION HI/CM ST MOTOR LONG 3.75
28716 - DRIVER INSERTION HI/CM ST MOTOR LONG 4.0
28717 - AR DUE CONE MORSE TAPER INSERTION DRIVER FOR MOTOR LONG
25945 - DRIVER ORTH MOTOR
26668 - DRIVER ORTH MOTOR LONG
28137 - DRIVER INSERTION Z MOTOR
27293 - DRIVER ORTH MOTOR SHORT
32743 - DRIVER TORX T6 MOTOR
32830 - DRIVER ORING MOTOR
32834 - DRIVER HEXAGONAL 1.17 MOTOR
32840 - DRIVER SQUARE 1.3 MOTOR
33066 - STRETTO IMPLANT INSERTION DRIVER 3.0 MOTOR
34663 - DRIVER STRETTO
34626 - DRIVER CM AR CA (M)
34627 - DRIVER CM AR CA (EL)
34659 - DRIVER HE 3.5 CA (M)
34660 - DRIVER HE 4.0 CA (M)
34661 - DRIVER HI 3.5 CA (M)
34662 - DRIVER HI 4.0 CA (M)
34719 - CUTTER EXTENSION

1. COMMERCIAL PRESENTATION FORMS OF THE PRODUCT

Implacil Surgical Instruments are supplied in individual packs.

The primary packaging consists of a blister produced with virgin PET plastic, and it is sealed with Tyvek.

List of items contained in the pack:

- 1 Surgical Instrument

2. INDICATION OF USE / PURPOSE

Implacil rotary surgical instruments are pieces made of standard stainless steel, offered in various sizes and lengths, used to assist in general dental surgery.

All the instruments that integrate this register have connection to equipment, so they are called rotators.

3. MODE OF PRODUCT USE

Implant DRIVER For Motor HE: Apply the necessary torque for the placement of HE implants.

Implant DRIVER For Motor HI: Apply the necessary torque for the placement of HI implants.

AR Morse Taper Insertion DRIVER For Motor / Driver CM AR CA (M) / Driver CM AR CA (EL): Apply the torque required for the placement of Morse AR Cone implants.

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IFU-007

Revisão: 10

DRIVER Motor for Graft Screw: Apply the necessary torque for the placement of bone graft screws.

Cutter Extension DRIVER: Increase the length of the cutter.

DRIVER Insertion HE Motor / Driver HE 3.5 CA (M) / Driver HE 4.0 CA (M): Apply the necessary torque for the placement of HE implants.

DRIVER Insertion HI Motor / Driver HI 3.5 CA (M) / Driver HI 4.0 CA (M): Apply the necessary torque for the placement of HI implants.

DRIVER Insertion HI/CM Motor: Apply the necessary torque for the placement of HI/CM implants.

DRIVER Insertion HI/CM ST Motor: Apply the necessary torque for the placement of HI/CM implants of lengths of 5 and 6mm.

DRIVER Insertion HE ST Motor: Apply the necessary torque for the placement of HE implants of lengths of 5 and 6mm.

DRIVER Insertion HI ST Motor: Apply the necessary torque for the placement of HI implants of lengths of 5 and 6mm.

AR Due Cone Morse Taper Insertion DRIVER For Motor: Apply the necessary torque for the placement of morse cone AR implants.

DRIVER ORTH Motor: Apply the necessary torque for the placement of ORTH implants.

DRIVER ORTH Motor Long: Apply the necessary torque for the placement of ORTH implants.

DRIVER ORTH Motor Short Apply the necessary torque for the placement of ORTH implants.

DRIVER Insertion Z Motor: Apply the necessary torque for placement of the Z abutment.

Driver Torx T6 Motor: Apply the necessary torque for the placement of the Torx screws.

Driver Oring Motor: Apply the necessary torque to place the Oring.

Driver Hexagonal 1.17 Motor: Apply the necessary torque for placing the hexagonal screws.

Driver Square 1.3 Motor: Apply the necessary torque to place the square head screws.

STRETTO IMPLANT INSERTION DRIVER 3.0 MOTOR / DRIVER STRETTO 3.0 CA (L): Apply the necessary torque to place the Stretto Implant.

34719-Cutter Extension: increase the length of the cutter

Installation Driver labeled "long" are 5mm longer than the length of their rod, and their use is identical to conventional size keys, whether or not it is a professional choice.

The rotary instruments of Implacil should be used to assist in dental surgeries. Previously they should be sanitized and sterilized according to the following techniques:

a) Cleaning:

The cleaning of a surgical instrument that has been in contact with tissues must be conducted in such a way that all parts of the item are exposed to the cleaning process.

To clean Implacil instruments, use only the following manual cleaning process.

Automated cleaning has not been validated. Do not use automated cleaning for Implacil instruments.

1. Initial rinse with cold water. The initial cold rinse is carried out in order to remove any gross contamination present in the item. Cold water is used, as temperatures above 60°C / 140 ° F will coagulate proteins, making it difficult to remove them from contaminated items.

2. Prepare the enzymatic detergent according to the recommendation *.

3. Dip the products in the prepared detergent solution and let it soak for at least 5 minutes. While soaking, brush the items with a soft bristle brush.

4. Rinse: after removing the instruments from the detergent solution, rinse thoroughly for at least 3 times (for at least 1 minute) under running water.

5. Visually inspect each product for visible dirt. If any visible dirt remains, repeat the cleaning process (steps 3 and 4) until no visible dirt is left.

* Recommendation for the cleaning step

Always use distilled, deionized or demineralized water for this procedure.

Never use saline solutions, especially sodium hypochlorite and saline, disinfectants, hydrogen peroxide or alcohol for cleaning or rinsing the cutters.

Never use steel wool or sponges and abrasive products, so that the cutters are not damaged.

Cleaning: Enzyme Detergent Composition: Protease, Lipase and Amylase enzymes

Dilution: 1mL for each liter of water;

Action time: 5 min (Manual);

Water temperature: 40 ° C to 55 ° C (Manual);

Note: The cleaning process must be conducted in such a way that all parts of the surgical instrument are exposed. Contamination remaining on an item after cleaning reduces the effectiveness of any subsequent decontamination or sterilization process.

Clean instruments as soon as possible after use. Do not allow blood and debris to dry on the instruments. If cleaning needs to be delayed, place groups of instruments in a container covered with enzyme solution or appropriate detergent to delay drying.

The instruments must be replaced as soon as the professional realizes their diminished effectiveness or degradation of the item.

b) Sterilization:

It is a procedure that aims at the total elimination of microorganisms, such as: viruses, bacteria, microbes, fungi (either in vegetative or sporulated form).

After the drying process, the instruments are sent to sterilization in an autoclave, using temperature and exposure time, according to the following method.

This product must be properly sanitized and sterilized before use. Please use for steam sterilization only the steam sterilization according to the following parameters:

Sterilization Parameters	
Temperature	132°C
Sterilization time	20 min
Drying time	20 - 60 min

Surgical instruments must be placed inside the container (perforated in the lid and at the bottom), or tray or box; provided that they can be prepared for the sterilization process. DO NOT open the autoclave to avoid rapid condensation; that is, let all the steam out first, making the Drying Cycle complete by itself. Thoroughly and periodically clean the autoclave, removing dirt and excess rust (iron oxide).

How to fit the motor key to the Contra Angle

1.Position the wrench to fit the contra angle

2.Insert the key into the contra angle until you hear a "click"

How to remove the Contra Angle motor key

1.Press the push button to remove the key

2.Pull the contra angle key

4. COMPOSITION

ASTM F899 Stainless Steel XM16

5. EXPIRY DATE

05 years from date of manufacture

6. STORAGE CONDITIONS

Keep in its original packaging in a place protected from direct sun and rain.

The product in question is not thermosensitive and does not show any variation in environments with high or low levels of humidity, until it can be sterilized by autoclaving. It is nevertheless recommended that the product is maintained between 15 ° and 35 ° C and at a humidity between 45% and 80%, to ensure stability of the package.

7. TRANSPORT CONDITIONS

The instruments should be transported in a clean and dry place at room temperature. The transport should be performed appropriately to avoid falls and damages in their original pack.

8. HANDLING CONDITIONS

The handling should be performed with the due cares to avoid falls and damages, it must be performed on shelves and counters, according to the activity that is being executed.

9. WARNINGS

Maximum care must be taken in the cleaning, sterilization, accommodation (storage), transport and handling processes of the product to avoid mechanical shocks that change their anatomy and compromise their function. Check the functioning of the product before initiating surgery. It should not be used if it has anatomical changes.

10. PRECAUTIONS

Improper maintenance, cleaning and sterilization may render the instruments unsuitable for its intended use, cause corrosion, disassembly, distortion and / or breakage, or cause injury or infection to the patient or operating personnel.

11. CONTRAINDICATIONS

Not applicable

12. ADVERSE EFFECTS

Not applicable

13. WARRANTY TERM

IMPLACIL DE BORTOLI - MATERIAL ODONTOLÓGICO S.A. guarantees the owner of this product against any material or manufacturing defect that may appear on it within 365 days, counting from the date of purchase by the first buyer. The product will have indefinite validity if transported, stored and packed suitably according to this User Manual. Note: the warranty does not cover wear from product use.

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