**PORTUGUÊS**

KIT MASTER CONICO VELOCE HE HI CM

**INGLÊS**

MASTER CONICO VELOCE HE HI CM KIT

**ESPAÑHOL**

KIT MASTER CONICO VELOCE HE HI CM

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## PRODUCT

Technical Name: Instrumental Kit

Commercial Name: MASTER CONICAL VELOCE HE HI CM KIT

## ATTENTION!!!

**Do not use if the packaging is broken or damaged.**

**Non-Sterile Product. Sterilization is required before use.**

**Product subject to reprocessing.**

## Commercial Models:

304000100 - MASTER CONICO CASE - VELOCE HE HI CM

29317 - HELICAL SPEAR DRILL 2.0

22651 - SUPER CUTTER DRILL 2.0

19651 - CUTTER EXTENSION DRIVER

19743 - MEDIUM RATCHET EXTENDER

18883 - TUNEL CHECK 2/3

19644 - CALIBRATED TUNNEL CHECK 2/3

25119 - TAPERED TUNNEL CHECK 3.5

25126 - TAPERED TUNNEL CHECK 4.0

25133 - TAPERED TUNNEL CHECK 5.0

25121 - REAMER LONG 3.5 - 07 A 13

25123 - REAMER LONG 4.0 - 07 A 13

18623 - DRIVER DIGITAL N° 01

18685 - DRIVER HEALING ABUTMENT

4057263505 - TAPERED DRILL 3.5X05

4057263507 - TAPERED DRILL 3.5X07

4057263509 - TAPERED DRILL 3.5X09

4057263511 - TAPERED DRILL 3.5X11

4057263513 - TAPERED DRILL 3.5X13

4057263515 - TAPERED DRILL 3.5X15

4057264005 - TAPERED DRILL 4.0X05

4057264007 - TAPERED DRILL 4.0X07

4057264009 - TAPERED DRILL 4.0X09

4057264011 - TAPERED DRILL 4.0X11

4057264013 - TAPERED DRILL 4.0X13

4057264015 - TAPERED DRILL 4.0X15

4057264505 - TAPERED DRILL 4.5X05

4057264507 - TAPERED DRILL 4.5X07

4057264509 - TAPERED DRILL 4.5X09

4057264511 - TAPERED DRILL 4.5X11

4057264513 - TAPERED DRILL 4.5X13

4057264515 - TAPERED DRILL 4.5X15

4057265005 - TAPERED DRILL 5.0X05

4057265007 - TAPERED DRILL 5.0X07

4057265009 - TAPERED DRILL 5.0X09

4057265011 - TAPERED DRILL 5.0X11

4057265013 - TAPERED DRILL 5.0X13

4057265015 - TAPERED DRILL 5.0X15

31185 - SCALED CYLINDRICAL MILLING DRILL 2.8MM

31187 - SCALED CYLINDRICAL MILLING DRILL 3.3MM

31189 - SCALED CYLINDRICAL MILLING DRILL 3.7MM

31191 - SCALED CYLINDRICAL MILLING DRILL 4.3MM

30330FC - TAPERED DRILL 3.0

24407 - TORQUE WRENCH 20-80 NCM

24708 - MEASUREMENT PROBE

24112 - DRIVER IMPLANT INSERTER HE 3.5 TI

23139 - DRIVER IMPLANT INSERTER HE 4.0 TI

25089 - DRIVER IMPLANT INSERTER FOR MOTOR HE 3.5

25096 - DRIVER IMPLANT INSERTER FOR MOTOR HE 4.0

17787 - DRIVER MEDIUM INSERTION 3.3 HI MEDIA

17770 - DRIVER MEDIUM INSERTION 3 75 HI

20152 - DRIVE IMPLANT INSERTER FOR MOTOR HI 3 5

24693 - DRIVE IMPLANT INSERTER FOR MOTOR HI 4 0

34622 - DRIVER CM AR TO (M)

34626 - DRIVER CM AR CA (M)

27212 - MINI DRILL SPEAR 2.0X05

## 1- INDICATION OF USE AND PURPOSE

The MASTER CONICO VELOCE HE HI CM KIT is indicated to assist dental surgeons in performing surgeries using Implacil Implants (a separate ANVISA-registered product, sold separately), which perform bone drilling and implant placement.

This KIT consists of Drill, Mini Drill Spear, Extension Driver, Tunnel Check 2/3, Drill, Torque Wrench, Measurement Probe, Tunnel Check TaperedaAnd Reamer.

## 2- HOW TO USE

PRODUCT FOR DENTAL USE ONLY

NON-STERILE PRODUCT - AUTOCLAVABLE

STERILIZE BEFORE USE

To install an Implacil implant (a separate ANVISA-registered product, sold separately), the MASTER CONICO VELOCE HE HI CM KIT must be used for this technique. To do so, a sequence of burs—available in the KIT, which is the subject of this petition—must be used to perform the bone drilling. The procedure is as follows: prepare the recipient bed, select the burs according to its diameter, and proceed with the drilling.

Attach the bur to the contra-angle handpiece. KIT the motor to a speed of 600 to 1200 rpm. Irrigation can be manual or automatic. With the bur rotating, the bone drilling is performed respecting the circumferential markings according to the planned implant. To place the implant, use the torque wrench and placement wrench according to the chosen implant, with a maximum torque of 60 Ncm.

The items in this KIT must be previously sanitized and sterilized using the following techniques:

a) Cleaning:

Cleaning must be carried out in such a way that all parts of the item - products that make up the case - are exposed to the cleaning process.

To clean the Implacil MASTER CONICO VELOCE HE HI CM KIT, use only the following manual cleaning process.

Automated cleaning has not been validated. Do not use automated cleaning for Implacil cases.

1. Initial rinse with cold water. The initial cold rinse is carried out in order to remove any gross contamination present on the items. Cold water is used, as temperatures above 60°C/140°F will coagulate proteins, making them difficult to remove from contaminated items.
2. Prepare the enzymatic detergent according to the recommendation\*.
3. Submerge the products in the prepared detergent solution and let soak for at least 5 minutes. While soaking, brush the items with a soft-bristled brush.
4. Rinse: After removing the cases from the detergent solution, rinse them intensively at least 3 times (for at least 1 minute) under running water.
5. Visually inspect each product for visible soil. If any visible soil remains, repeat the cleaning process (steps 3 and 4) until no more visible soil remains.

\*Recommendation for the cleaning step:

Always use distilled, deionized or demineralized water for this procedure.

Never use saline solutions, especially sodium hypochlorite and saline solution, disinfectants, hydrogen peroxide or alcohol to clean or rinse the burs.

Never use steel wool or sponges or abrasive products, so that the burs are not damaged.

Cleaning: Enzymatic Detergent - Composition: Protease, Lipase and Amylase enzymes

Dilution: 1mL for each liter of water;

Action time: 5 min (Manual);

Water temperature: 40°C to 55°C (Manual);

Note: The cleaning process must be conducted in such a way that all parts of the cases and their components are exposed. Contamination remaining on an item after cleaning reduces the effectiveness of any subsequent decontamination or sterilization process.

Clean the cases and their components as soon as possible after use. Do not allow blood and residue to dry on cases. If cleaning needs to be postponed, place the case in a covered container with an enzymatic solution or appropriate detergent in order to delay drying.

The cases should be replaced as soon as the professional notices their reduced effectiveness or degradation. According to the manufacturer of the raw material, the

cases withstand more than 1,000 steam sterilization cycles at 134 °C without any significant loss of properties.

b) Sterilization:

This product must be properly sanitized and sterilized before use. Please use only steam sterilization for sterilization according to the following Sterilization Parameters:

Temperature: 132°C

Sterilization time: 20 min

Drying time: 20 - 60 min

DO NOT open the autoclave to avoid rapid condensation; that is, let all the steam escape first, allowing the Drying Cycle to complete itself. Thoroughly and periodically clean the autoclave, removing dirt and excess rust (iron oxide).

### 3.FORMS OF COMMERCIAL PRESENTATION OF THE PRODUCT

The MASTER CONICO VELOCE HE HI CM KIT, reference code "304000101", is sold non-sterile and reusable, packaged in an autoclavable case (BASE, LID, BOTTOM, AND RUBBER). The packaging consists of a cardboard box.

It consists of the following items, supported on a suitable autoclavable tray: Drill, Mini Drill Spear, Extension Driver, Tunnel Check 2/3, Drill, Torque Wrench, Measurement Probe, Tunnel Check Tapered And Reamer.

This instruction for use will be available in electronic format on the Implacil website (<https://implacilosstem.com.br/>) in accordance with current legislation on the provision of instructions for use in non-printed format for health products – RDC No. 751/2022. The printed instruction for use will be available to the customer who requests it, and will be sent free of charge. The printed instruction for use can be requested through the Customer Service – 0800.779.0011, and will be sent at no additional cost.

To locate it, check the "IFU" number and the version of the instruction that is indicated on the label of the product purchased. To read the instructions for use on the website, the Adobe Acrobat Reader program is required.

User Alert: Always observe the correlation of the version of the instructions for use indicated on the label of the product purchased, as made available by Implacil.

### 4. PHYSICAL PRINCIPLE AND FUNDAMENTALS OF THE PRODUCT'S TECHNOLOGY, APPLIED TO ITS OPERATION AND ACTION:

The MASTER CONICO VELOCE HE HI CM KIT was created for surgeries using Implacil Implants (a separate ANVISA-registered product, sold separately), providing dental surgeons with greater precision in the placement of certain implants.

## **5. COMPOSITION**

The kit's raw material consists of PSU. The instruments that make up the kit include:

- Drill, Mini Drill Spear, Extension Driver, Tunnel Check 2/3, Drill, Torque Wrench, Measurement Probe - Stainless Steel
- Tunnel Check Tapered - Titanium
- Reamer – Titanium

## **6. VALIDITY PERIOD**

Undetermined

## **7. SPECIAL CONDITIONS FOR TRANSPORTATION, STORAGE AND HANDLING OF THE PRODUCT:**

Keep in its original packaging in a place protected from direct sunlight and rain.

The product in question is not heat-sensitive and does not present any variation in environments with high or low levels of humidity, which is why it can be sterilized in an autoclave.

Even so, it is recommended that the product be kept between 15° and 35° C and at a humidity between 45% and 80%, to ensure the stability of the packaging.

After use, the MASTER CONICO VELOCE HE HI CM KIT must be stored clean, decontaminated and completely dry.

The packaging in which the items are sterilized can provide an effective barrier to maintain the sterility of the item.

The Kit must be transported in a clean, dry place at room temperature.

Transportation must be done properly, to avoid falling and damage to its original packaging.

The utmost care must be taken in the process of cleaning, sterilization, storage, transportation and handling of the product to avoid mechanical shocks that alter its anatomy and impair its functionality.

Before starting the surgery, check that the product is working properly.

It should not be used if it presents anatomical alterations.

Exclusive use by dental professionals.

## **8. WARNINGS AND/OR PRECAUTIONS TO BE ADOPTED**

Exclusive use by dental professionals. The product is not sterile, and this procedure must be performed by the professional before any intervention on the patient.

Do not use it under any circumstances if the packaging is broken. The product is not sterile, and this procedure must be performed by the professional before any intervention on the patient.

Incorrect maintenance, cleaning and maintenance may render the instruments unsuitable for their intended use, cause corrosion, disassembly, distortion and/or breakage or even cause injury to the patient or operating team.

## **9. CONTRAINDICATIONS**

Not applicable

## **10. ADVERSE EFFECTS**

Not applicable

## **11. PRECAUTIONS IN CASE OF DISPOSAL**

To avoid environmental contamination or misuse of MASTER CONICO VELOCE HE HI CM KIT, when they are no longer used, they must be disposed of in hospital waste or in a location for potentially contaminated materials.

## **12. WARRANTY TERMS**

IMPLACIL OSSTEM - MATERIAL ODONTOLOGICO SA guarantees the owner of this product a guarantee against any material or manufacturing defect that may arise within 365 days, counting from the date of purchase by the first purchaser.

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