PORTUGUÊS

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INGLÊS

Laboratory Materials

ESPAÑHOL

Materiales de Laboratorio

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PRODUCT

Commercial Name: Laboratory Materials

ATTENTION!

Do not use if the packaging is broken or damaged.

Non-Sterile Product.

Commercial Models:

Calcinable Abutment (Plastic Uclas);

Calcinable O'ring for Positioning;

Plastic Copings;

Analog;

SCANCORP;

Clip Bar;

Prosthetic Selection Kit;

Irrigation Serum Connector;

See codes and descriptions in Annex I

1- INDICATION OF USE AND PURPOSE

The laboratory materials are indicated to aid in the making of single-tooth or multiple-teeth prosthetic element, according to description and application of each product

2- HOW TO USE

- Calcinable Abutment (Plastic Uclas):

Calcinable Component, used for casting, also known as plastic coping that fits directly over the laboratory model to be waxed in the ideal position and cast, becoming an abutment or customized metal structure and then applied over the implant;

- Best indication for making customized abutments;
- Versatile component that may be cemented or screwed, applied for overdenture, protocol and single-tooth or multiple-teeth elements;
- Anti-Rotational or Rotational;
- Installation Torque: 30Ncm;
- Driver for installation – Square Driver – no. 4 – 1.3 mm.

Single-use product.

This product should not be sterilized.

Models:

Plastic Ucla HE Anti-Rotational 3.5 / 4.0 / 5.0

Plastic Ucla HE Rotational 3.5 / 4.0 / 5.0

Plastic Ucla HE Anti-Rotational ST 4.0

Plastic Ucla HE Rotational 4.0

Plastic Ucla HI Rotational 3.5 / 4.0 / 5.0

Plastic Ucla HI RAR (with hexagon, with shoulder) 3.5 / 4.0 / 5.0

Plastic Ucla HI AR (with hexagon, without shoulder) 3.5 / 4.0 / 5.0

Plastic Ucla HI R (without hexagon, without shoulder) 3.5 / 4.0 / 5.0

Plastic Ucla UNII HE 3.5 M1.6 / M1.8 SP

Plastic Ucla UNII HE 4.0 / 5.0 SP

Plastic Ucla UNII HE AR 3.5 M1.6 / M1.8 SP

Plastic Ucla UNII HE AR 4.0 / 5.0 SP

Plastic Ucla UNII HI 4.0 ST SP

Plastic Ucla UNII HE AR 4.0 ST SP

Plastic Ucla UNII HI 3.5 / 4.0 / 5.0 SP

Plastic Ucla UNII HI R 3.5 / 4.0 / 5.0 SP

Plastic Ucla UNII HI AR 3.5 / 4.0 / 5.0 SP

Plastic Ucla UNII HI RAR 3.5 / 4.0 / 5.0 SP

Plastic Ucla UNII HI R 4.0 ST SP

Plastic Ucla UNII HI AR 4.0 ST SP

Plastic Ucla UNII HI RAR 4.0 ST SP

PLASTIC HE

PLASTIC HI

- Calcinable O'ring for Positioning:

Prosthetic component manufactured in acrylic, used for fixation of overdenture.

- Component with calcinable plastic individual ball head used for casting
- Cast in a laboratory over the bars with parallelometer, distributed in the ideal position correcting divergent implants;
- It is not used as a single-tooth element.

Single-use product.

This product should not be sterilized.

Model:

Calcinable O'ring for Positioning

- Plastic Copings: Component manufactured in polyacetel, used as a connecting link between the prosthetic abutment and the dental implant.

Single-use product.

This product should not be sterilized.

Models:

Plastic Cemented Rotational Coping 3.5 / 4.5

Plastic Cemented Anti-Rotational Coping 3.5 / 4.5

Plastic Screwed Rotational Coping 3.5 / 4.5

Plastic Screwed Anti-Rotational Coping 3.5 / 4.5

Plastic Micro Tapered Coping 3.5

Abutment Stretto Plastic Coping 3.0x4.8;

Plastic Coping Base T 3.5;

Abutment Stretto Plastic Coping 3.0x3.5 / 3.0x4.8;

Abutment Ideale Plastic Coping CM 3.3x4 C/Ar;

Abutment Ideale Plastic Coping CM 3.3x6 C/Ar;

Abutment Ideale Plastic Coping CM 4.5x4 C/Ar;

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IFU-020

Revisão: 10

Abutment Ideale Plastic Coping CM 4.5x6 C/Ar;
Plastic Coping Base T 3.5/ 4.5
Morse Taper Abutment Plastic Coping AR C 3.5X4 / 4.5X4 / 4.5X6 / 3.5X6
Morse Taper Abutment Plastic Coping AR 3.5X4 / 4.5X4 / 4.5X6 / 3.5X6
Morse Taper Abutment Coping For Angled 3.5X4 / 4.5X4 / 4.5X6 / 3.5X6
Abutment Hexagonal Plastic Coping
Rotational Morse Taper (CM) Abutment Plastic Coping C 3.5X4 / 4.5X4 / 4.5X6 / 3.5X6
Aesthetic Conical Plastic Coping W/O HEX / W/ HEX
Mini Tapered Plastic Coping L11
Morse Taper Abutment Plastic Coping AR P 3.5X4 / 3.5X6 / 4.5X4 / 4.5X6
Rotational Morse Taper (CM) Abutment Plastic Coping P 3.5X4 / 3.5X6 / 4.5X4 / 4.5X6
Micro Tapered Plastic Coping

The rotational copings are indicated for multiple-teeth elements.

The anti-rotational copings are indicated for single-tooth elements.

Cemented Coping

Screwed Coping

Abutment Stretto
Plastic Coping

- **Analogs:** The analogs are manufactured in stainless steel and are used as a replica of the prosthetic abutments or implant, used in the prosthetic laboratory with the intention of simulating the position of the abutment / implant in the mouth of the patient through mold allowing the performance of prosthetic work.

This product should not be sterilized.

Models:

Analog HE 3.5 / 4.0 / 5.0

Analog HI 3.5 / 4.0 / 5.0

Tapered Aesthetic Anti-Rotational Analog

Tapered Aesthetic Rotational Analog

Mini Tapered Rotational Analog

Morse Taper Abutment Analog 3.5X4 / 3.5X6 / 4.5X4 / 4.5X6

Morse Taper Angled Abutment Analog 3.5X4 / 3.5X6 / 4.5X4 / 4.5X6

Morse Taper Implant Analog

Classic Analog 3.25 / 3.75 / 4.25

Hexagonal Abutment Analog

Micro Tapered Analog

ANALOG MINI CONICAL

Ideale Morse Taper Abutment Analog 3.3X4

Ideale Morse Taper Abutment Analog 3.3X6

Ideale Morse Taper Abutment Analog 4.5x4

Ideale Morse Taper Abutment Analog 4.5x6

ANALOG ABUTMENT STRETTO

ANALOG IMPLANT STRETTO

Ideale Morse Taper
Abutment Analog

Mini Tapered

Tapered Aesthetic Anti
Rotational

Tapered Aesthetic
Rotational

Rotational

Abutment CM

CM Implant

Classic

- **The digital laboratory analogs / plaster (hybrid):** should be used as replicas of dental implants, prosthetic components or prosthetic intermediates, to be inserted in a 3D model printed, milled or applied to plaster, in order to duplicate the location, orientation and restorative platform of the implant, or prosthetic component applied to patients.

Through digital files, they offer high precision for the execution of laboratory processes in CAD (Computer Aided Design) and CAM (Computer Aided Machine). Due to its hybrid format, it can also be used in common laboratory plaster fixation processes.

- 31509 ANALOG D/G CM AR
- 31510 ANALOG D/G HE 3.5 AR
- 31511 ANALOG D/G HE 4.0 AR
- 31512 ANALOG D/G HE 5.0 AR
- 31513 ANALOG D/G HI 3.5 AR

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IFU-020

Revisão: 10

- 31514 ANALOG D/G HI 4.0 AR
- 31515 ANALOG D/G HI 5.0 AR
- 31518 ANALOG D/G IDEALE 3.3X4 AR
- 31519 ANALOG D/G IDEALE 3.3X6 AR
- 31520 ANALOG D/G IDEALE 4.5X4 AR
- 31521 ANALOG D/G IDEALE 4.5X6 AR
- 31517 ANALOG D/G TAPERED AESTHETIC AR
- 31516 ANALOG D/G TAPERED AESTHETIC R

- 30880 ANALOG D/G MINI TAPERED R

- 31522 ANALOG D/G MICRO TAPERED R

- 1063511MCC ANALOG REVERSE DIGITAL MICRO TAPERED

- 1064811MNC ANALOG REVERSE DIGITAL MINI TAPERED

- **SCANCORP:** Scancorp is used in combination with the T Base abutments;
For scanning in the CAD CAM system.

Models:

Scancorp 3.5

Scancorp 4.0

- **Clip Bar** Used for retention of removable prostheses, on two or more cylindrical bars cast using plastic clips fixed in the prosthesis.

- Clip type plastic component that adapts to the bar for overdentures, used as a retaining system in which the implants are joined together by a metal bar;
- It can also be used in combination with the Calcinable O'ring (Mixed System);
- Indicated for the mandible and maxilla.

Models:

Clip Bar - Kit

Clip Bar - Cylindrical

Clip Bar - Straight

Clip Bar - 35°

Clip Bar - 70°

- **Prosthetic Selection Kit:**

FOR USE IN LABORATORY PLANNING

- The most complete selection kit for Morse Taper components: helps in the selection of the transmucosal implant, type of component, angulation, diameter and height of the element to be used.
- The only system with the option of 3 (three) combinations of the same collars, which helps in the planning for multiple-teeth elements;
- Easy to handle;
- High precision;
- Made in aluminum.

See Annex II – for further handling and coupling information.

3- COMMERCIAL PRESENTATION FORMS OF THE PRODUCT

The products contemplated in this instruction for use are presented in individual packaging.

The packaging consists of a blister sealed with Tyvek®.

List of items contained inside the packaging

- 01 Laboratory Material
- 03 Tracking Labels* with product identification for obligatory attachment: on the clinical record, on the document to be submitted to the patient, and on the invoice documentation

*Note : Items supplied with the tracking label:

- Calcinable Abutment (Plastic Uclas)
- Calcinable O'ring for Positioning
- Plastic Copings
- Plastic Uclas

This instruction for use will be available in electronic format on the Implacil website (<https://implacilosstem.com.br/>) in accordance with current legislation on the provision of instructions for use in non-printed format for health products – RDC No. 751/2022. The printed instruction for use will be available to the customer who requests it and will be sent free of charge. The printed instruction for use can be requested through the Customer Service – 0800.779.0011, and will be sent at no additional cost. To locate it, check the "IFU" number and the version of the instruction that is indicated on the label of the product purchased. To read the instructions for use on the website, the Adobe Acrobat Reader program is required.

User Alert: Always observe the correlation of the version of the instructions for use indicated on the label of the product purchased, as made available by Implacil.

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IFU-020

Revisão: 10

4- COMPOSITION

As raw materials for Laboratory Materials, we have:

- Stainless Steel – Analog;
- Aluminum (Morse Taper/CM Selection Kit) – Collar; Abutment; Straight Abutment; Mini Tapered; Esthetic Conical;
- PEEK – SCANCORP;
- POLYACETAL – Plastic Coping; Plastic/Classic Ucla; Castable O-RING for positioning; Clip Bar;
- ULTEM® – Case

5- VALIDITY PERIOD

5 years from the date of manufacture

6- SPECIAL CONDITIONS FOR TRANSPORTATION, STORAGE AND HANDLING OF THE PRODUCT:

Keep at room temperature, in a cool place, away from sunlight and heat. Avoid exposure to heat and excess humidity.

The laboratory materials should be transported in a clean and dry place at room temperature.

The transport should be performed appropriately to avoid falls and damages in their original pack.

Do not use the product if the packaging is damaged.

Maximum care must be taken in the accommodation (storage), transport and handling processes of the product to avoid mechanical shocks that change their anatomy and compromise their function.

The product should not be used if it has anatomical changes. When handling the materials, use extreme caution in order to avoid injuries.

7- WARNINGS AND/OR PRECAUTIONS TO BE ADOPTED

Exclusive use by the dental professional.

Incorrect maintenance, cleaning and maintenance can make the instruments inappropriate for their intended use, cause corrosion, dismounting, distortion and/or break or even cause injuries on the patient or operating team.

Below is a list of potential complications:

- Injury or pain in the soft tissues, in case of incorrect use or breakage of the instruments.
- Infection, if the instrument is not conveniently clean and sterilized.
- Damages to the nerves or nearby organs as a result of slide or bad positioning of a damaged instrument.
- Damages caused by instruments used to screw, due to excess forces that occur during use.
- Increase in the time of operation due to dismounting of the instruments during surgery.
- Due to the necessary mechanical functionalities, majority of the instruments (drills) are made in non-implantable materials. In case an instrument breaks, no fragment should remain inside the patient because it can cause post-operative complications, such as allergies, infections or complications of the biological kind, associated to the

release of metallic components, possibly requiring future intervention.

To avoid environmental contamination or undue use of the Implacil non-articulated cutting surgical instruments, when these are unusable, the instruments should be disposed in the appropriate place, i.e., in a place for potentially contaminated materials.

8- CONTRAINDICATIONS

Not applicable

9- ADVERSE EFFECTS

Not applicable

10. PRECAUTIONS IN CASE OF DISPOSAL

To avoid environmental contamination or misuse of Laboratory Materials, when they are no longer used, they must be disposed of in hospital waste or in a location for potentially contaminated materials.

11. WARRANTY TERMS

IMPLACIL OSSTEM - MATERIAL ODONTOLOGICO SA guarantees the owner of this product a guarantee against any material or manufacturing defect that may arise within 365 days, counting from the date of purchase by the first purchaser.

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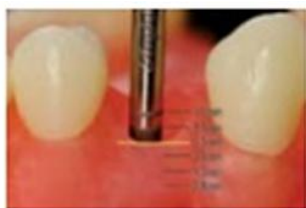
PASSO A PASSO/ STEP BY STEP/ PASO A PASO:

O Túnel Check aplicado para medição, conforme ilustração acima, mostra que temos a profundidade de 3.5 mm até o limite gengival / The Tunnel Check applied for measurement, as illustrated above, shows that we have the depth of 3.5 mm to the gingival limit / Túnel Check aplicado para medición, según la ilustración anterior, muestra que tenemos la profundidad de 3.5 mm hasta el límite de la encía

Descontado o valor de 2 mm selecionado então a cima 15 mm/ Discounting the value of 2 mm selecting the collar 1.5 mm/ Descontado el valor de 2 mm seleccionados entonces arriba de 15 mm

Cinta aplicada, observação do nível gengival ideal/ Collar applied, observation of the ideal gingival level / Cinta aplicada, observación del nivel ideal de la encía

Seleção e prova de altura do munhão / Selection and proof of abutment height / Selección y prueba de altura del muñón

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EN Planning is no doubt one of the most important phases of the treatment and a success determining factor of the prosthesis over implants.

With the Selection kit we are able to analyze:

- Diameter;
- Abutment Height;
- Type of cemented or screwed system;
- Straight or angled.

The following criteria must be observed when planning a treatment that involves prosthesis over implants, namely:

- Ideal position of the crown in the arch;
- Ideal position of the implant (taking as reference the already established position of the crown).
- Selection of the most suitable component to obtain the best crown/implant connection result.

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